



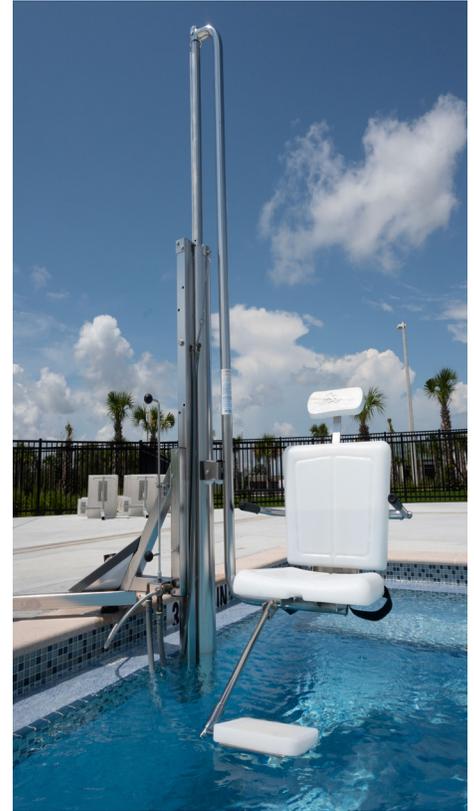


Valve Replacement for Water Powered Lifts

1. Disconnect water supply line from valve.

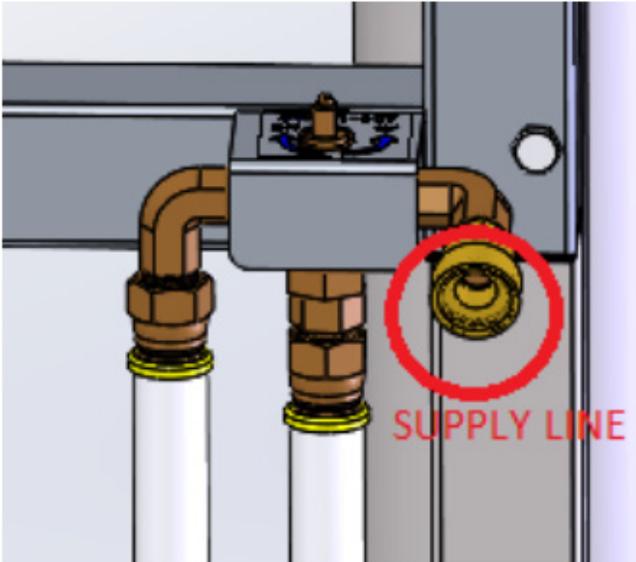


2. Remove handle from valve. And remove Valve body from attachment bracket.

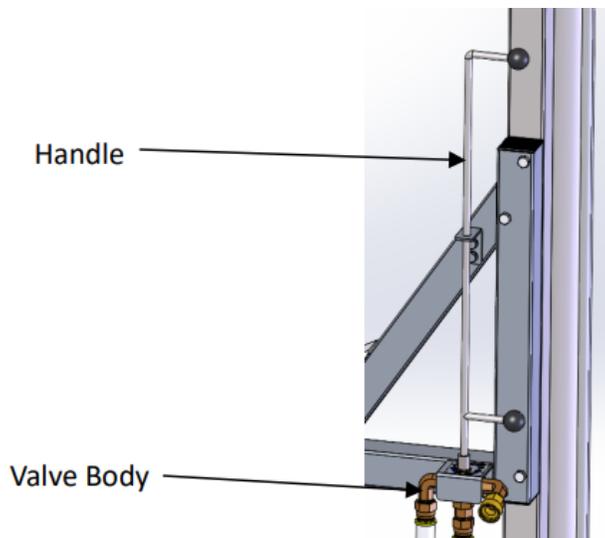




3. Disconnect Influent line and fitting from cylinder. Replace with new fitting and Influent hose from valve.

Influent Line and Fitting

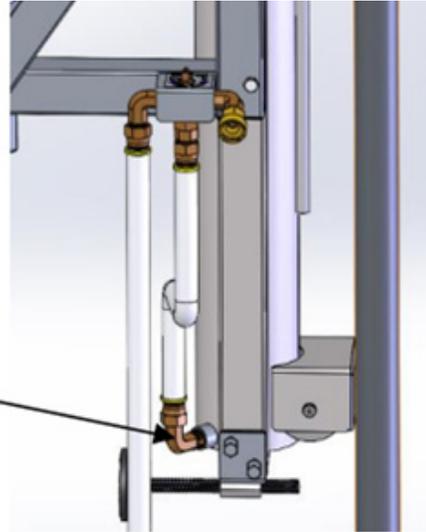

4. Remove Small hose and fitting from cylinder. Replace with new fitting and hose.

Small Hose and Fitting

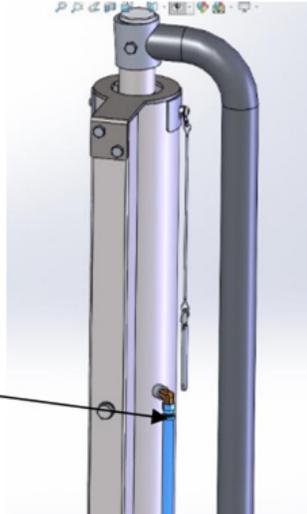

5. Reinstall Valve body onto attachment bracket.
6. Reinstall Handle onto Valve body.
7. Install hose gasket into source fitting and reattach supply line.

TROUBLESHOOTING:

Any questions you have can be answered by the Spectrum Customer Service at 406.532.6321 or emailing them at service@spectrumproducts.com.