

MAINTENANCE CHECKLIST

ADA POOL LIFTS

Maintenance Task to Perform	Daily	Weekly	Monthly
Charge battery overnight*	X		
Rinse lift with non-chlorinated water and wipe dry with soft, non-abrasive cloth**		X	
Clean with Spectra-Clean™ to passivate the stainless steel		X	
Clean and Apply Dielectric Grease to Control Box and Battery		X	
Test lift for full operation***		X	
Visual Inspection—check for visible damage and corrosion, and to ensure that all hardware is snug			X
Lubricate all pivot points with Light 3-in-1 Oil (WD-40)			X
Grease Lower Bearing Assembly with FDA-approved Grease using a grease gun through oil zerk			X

IMPORTANT NOTES:

*When not in use, remove the battery from the lift and store it on the charger to maintain the battery's charge.

**Use only fresh water to wash your lift, not pool water!

**NOTE: Do not use high pressure power wash to rinse--damage can occur!

***DO NOT manually move the lift left and right by pushing it (unless instructed otherwise by a Spectrum technician.)

Please make copies for regular use in your facility

Need help? Please contact our friendly customer service team: techsupport@spectrumproducts.com

or call us at 800-791-8056