


INSTRUCTIONS –TOP COLLAR REPLACEMENT (Lolo/Gallatin)


- 1) Remove lift from pool
Remove seat arm from piston rod
Remove water supply hose.

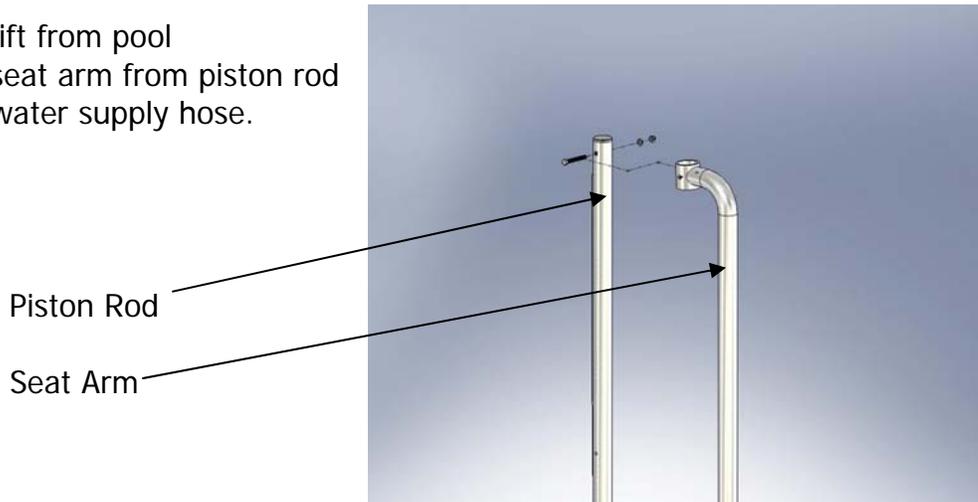

- 2) Note: Piston Rod not shown for clarity but will be in this assembly.

Pull piston rod up as arrow shows

Remove 2 large Hex head bolts from square tube, and then remove the 4 small hex head bolts from the top of the cylinder.

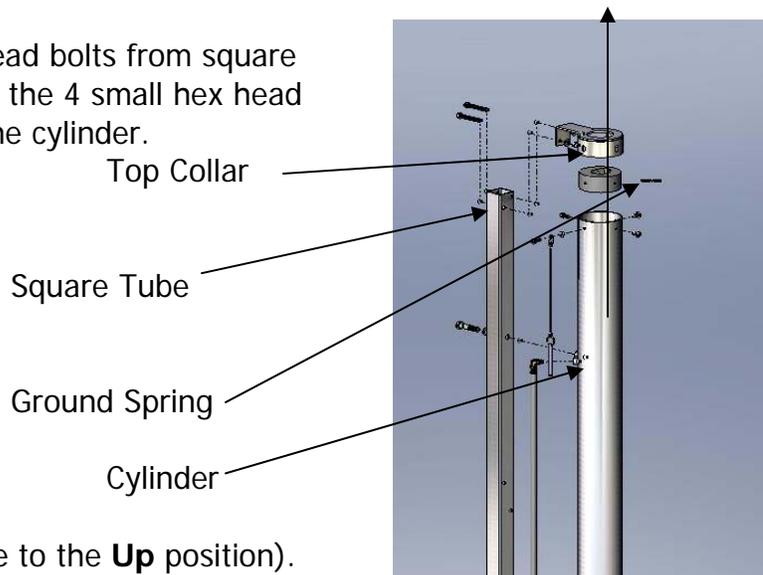


- 3) Open Valve (Turn valve to the **Up** position).

Pull on piston rod until the Top collar just comes out of cylinder tube.
Slide top collar off of piston rod.


4) Slide new Top Collar onto piston rod making sure to put the grounding spring in as shown on the last picture.

IMPORTANT - Also note the orientation of the top collar in relation to the square tube .

Piston Rod not shown for clarity.

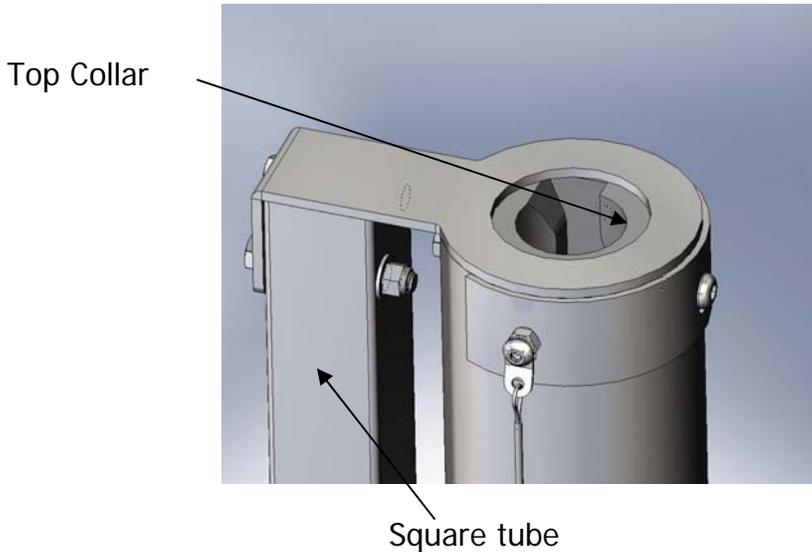

Replace all hardware and run lift a few times for break in without weight, look for proper operation.

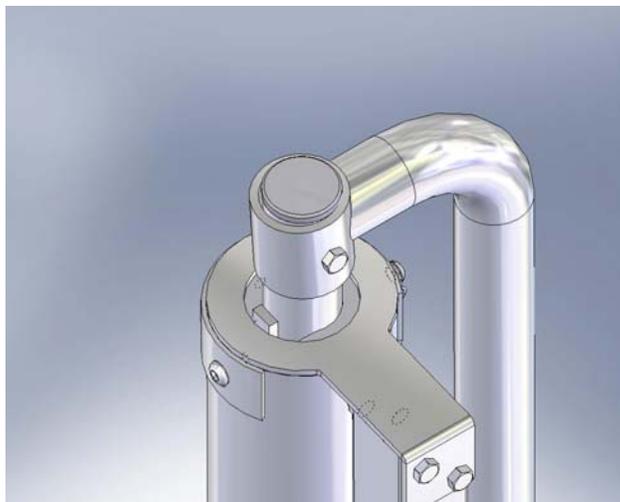

How the lift should look when finished.